



Innovation in Gynaecology



## LiNA Xcise™


Cordless Morcellator  
Improved Safety  
Reducing procedure time



Adjustable coreguard



Reducer cap

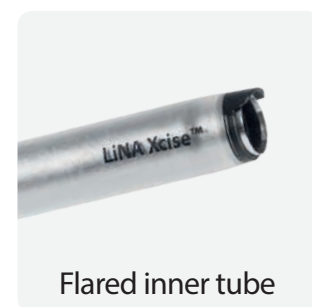

Flared inner tube

The LiNA Xcise™ is a cordless fully disposable morcellator intended for tissue morcellation during laparoscopic gynaecological procedures. The unique design of the LiNA Xcise™ provides safe and easy control during hysterectomy and myomectomy.

# LiNA Xcise™ Cordless Morcellator

## Simplicity

### Single use device that is completely ready to use

- Cordless and no assembly required**

The morcellator can be used right out of the box and as it is cordless there is no need for assembly to driver box or cable. The LiNA Xcise™ is therefore quick, simple and easy to use and facilitates unlimited mobility of the OR staff.

- Fully disposable**

The LiNA Xcise™ is RoSH compliant and fully disposable in regular hospital waste.

- Power**

The LiNA Xcise™ has a higher torque and runs at 1,000RPM to morcellate through the most fibrotic tissue (Largest reported uterus morcellated by LiNA Xcise was 2930gr).



## Innovation

### Easy and effective morcellation

- Flared cutting tube**

Cutting tube on The LiNA Xcise™'s flares out to a larger diameter as tissue is pulled through, which reduces drag, provide long tissue strips and reduces tissue rotation.

- Inner bevel blade design**

The LiNA Xcise™ features a large inner bevel blade design preventing the blade from going dull from inadvertent contact with the tenaculum.

- Adjustable coreguard**

The coreguard on the cutting tube can be adjusted and rotated 360° according to the hand position ensuring optimal peeling effect.

- 5mm reducer cap**

The reducer cap has a simple snap-on design for quick exchange and optimal use of 5mm instrument.



## Safety

### Full control during morcellation

- Three-step positioning**

The LiNA Xcise™ has three blade positions to secure an optimal and safe morcellation;

"Safety Guard" – Blade fully retracted into the cannula for patient safety

"Cut 1" for surface peeling

"Cut 2" for coring



### Ordering information:

LiNA Xcise™

REF.	Product	Packing
MOR-1515	LiNA Xcise – 15mm morcellator with obturator – single use	2x boxes of 6, sterile
MOR-1515-1	LiNA Xcise – 15mm morcellator with obturator – single use	2x boxes of 6, sterile, individually packed
MOR-1515-6	LiNA Xcise – 15mm morcellator with obturator – single use	1x box of 6, sterile