




LiNA SeaStar™ and LiNA Skin Hook™

Retractor ring and skin hook


Complete system for retracting soft tissue from the operative surface




Prepare and drape the surgical area in usual manner. Apply the LiNA SeaStar according to the following guidelines:

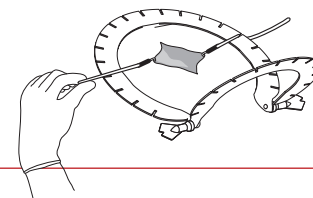

1. Place the LiNA SeaStar around the initial incision and insert the LiNA Skin Hook in directly opposing quadrants with equal tension.

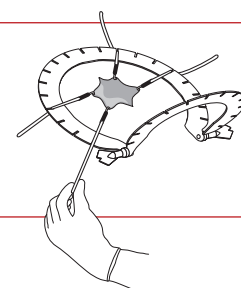

2. As dissection progress, move the position of the LiNA Skin Hook into the deeper layers of fascia. You may then add new hooks or reposition those already in place to achieve full balanced retraction from any angle.

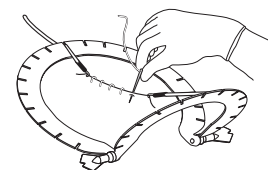

3. To ease closing, maintain constant opposing tension.

Order information:

LiNA SeaStar™ and Skin Hook™



REF. single use, sterile	REF. reusable, non-sterile	Product	Packing, single use	Packing, reusable
4000-S-Q		LiNA SeaStar – squared. 18cm x 18cm. 62g. (7 in x 7 in 2.2 oz.)	Box of 5	
4000-S-SS	4000-R-SS	LiNA SeaStar – small/small. 29cm x 19cm. 75g. (11 in x 7 in 2.6 oz.)	Box of 5	Box of 1
4000-S-SL	4000-R-SL	LiNA SeaStar – small/large. 35cm x 22cm. 94g. (14 in x 9 in 3.3 oz.)	Box of 5	Box of 1
4000-S-VV	4000-R-VV	LiNA SeaStar – v-shape/v-shape. 23cm x 19cm. 53g. (9 in x 7 in 1.9 oz.)	Box of 5	Box of 1
4000-S-VS	4000-R-VS	LiNA SeaStar – v-shape/small. 26cm x 19cm. 64g. (10 in x 7 in 2.3 oz.)	Box of 5	Box of 1
4000-S-VL	4000-R-VL	LiNA SeaStar – v-shape/large. 32cm x 22cm. 83g. (13 in x 9 in 2.9 oz.)	Box of 5	Box of 1
	4000-R-SRW	LiNA SeaStar screws.		Box of 10
SH-110		LiNA Skin Hook – small sharp hook.	2 pieces in a pouch, box of 5 pouches	
SH-120		LiNA Skin Hook – small blunt hook.	2 pieces in a pouch, box of 5 pouches	
SH-220		LiNA Skin Hook – large blunt hook.	2 pieces in a pouch, box of 5 pouches	



LiNA Medical ApS

• Formervangen 5
DK-2600 Glostrup
Denmark

• Tel.: +45 43 29 66 66
Fax: +45 43 29 66 99
info@lina-medical.com

• www.lina-medical.com
CVR: 10054974

LiNA SeaStar™ and LiNA Skin Hook™

The LiNA SeaStar and LiNA Skin Hook is a complete system for retracting both delicate and solid tissue. The flexible design of the system allows easy adjustment and repositioning of the LiNA SeaStar during the surgical procedures and provides optimal exposure of the surgical site.

Safe

Self-retaining

- The LiNA SeaStar and LiNA Skin Hook is a self-retaining system freeing the hands of the surgeon and the surgical assistant to handle other tasks in the OR.


Functional

Broad range of LiNA SeaStar retractor rings


- LiNA SeaStar**
The LiNA SeaStar is provided in a broad range of shapes and sizes which can easily be adjusted to fit various anatomical areas, thereby facilitating different surgical applications e.g. vaginal and rectal surgery.
- LiNA Skin Hook**
By securing the LiNA Skin Hook on to the LiNA SeaStar the exposure of the incision is increased. The flexible and elastic hook can be adjusted during the procedure to create optimum tension. The LiNA Skin Hook is available with 3 different stainless steel hooks.



The LiNA SeaStar is available both as single use (S) and reusable (R), except for the squared model which can only be ordered as single use.

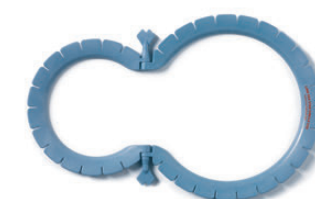

REF 4000-S-SL
REF 4000-R-SL

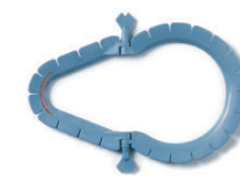


REF 4000-S-VS
REF 4000-R-VS




REF 4000-S-SS
REF 4000-R-SS



REF 4000-S-VV
REF 4000-R-VV

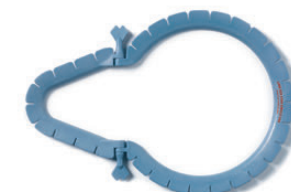

REF 4000-S-VL
REF 4000-R-VL

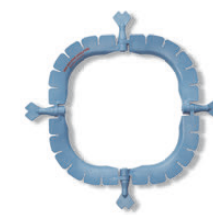

REF 4000-S-Q



REF SH-110
Small sharp hook



REF SH-120
Small blunt hook



REF SH-220
Large blunt hook